**medi GmbH & Co. KG**  
**Medicusstraße 1**  
**D-95448 Bayreuth**  
**Germany**  
**T +49 921 912-0**  
**F +49 921 912-780**  
**ortho@medi.de**  
**www.medi.de**

**medi Australia Pty Ltd**  
**83 Fennell Street**  
**North Parramatta NSW 2151**  
**Australia**  
**T +61-2 9890 8696**  
**F +61-2 9890 8439**  
**sales@mediaustralia.com.au**  
**www.mediaustralia.com.au**

**medi Austria GmbH**  
**Adamgasse 16/7**  
**6020 Innsbruck**  
**Austria**  
**T +43 512 57 95 15**  
**F +43 512 57 95 15 45**  
**vertrieb@medi-austria.at**  
**www.medi-austria.at**

**medi Belgium bvba**  
**Posthoornstraat 13/1**  
**3582 Koersel**  
**Belgium**  
**T: + 32-11 24 25 60**  
**F: +32-11 24 25 64**  
**info@medibelgium.be**  
**www.medibelgium.be**

**medi Brasil**  
**Rua Cristovan de Vita, n.º 260, Un. 30,**  
**Centro Logístico Raposo Tavares,**  
**Vila Camargo**  
**Vargem Grande Paulista - SP**  
**CEP 06730-000**  
**Brasil**  
**T: +55-11-3500 8005**  
**sac@medibrasil.com**  
**www.medibrasil.com**

**medi Canada Inc / médi Canada Inc**  
**597, Rue Duvernay, Verchères**  
**QC Canada J0L 2R0**  
**T: +1 450-583-3317 / +1 800 361 3153**  
**F: +1 888-583-6827**  
**service@medicanada.ca**  
**www.medicanada.ca**

**MAXIS a.s.,**  
**medi group company**  
**Slezská 212/713**  
**120 00 Prague 2**  
**Czech Republic**  
**T: +420 571 633 510**  
**F: +420 571 616 271**  
**info@maxis-medica.com**  
**www.maxis-medica.com**

**medi Danmark ApS**  
**Vejlegardsvej 59**  
**2665 Vallensbaek Strand**  
**Denmark**  
**T +45-70 25 56 10**  
**F +45-70 25 56 20**  
**kundeservice@sw.dk**  
**www.meditandmark.dk**

**medi Bayreuth Espana SL**  
**C/Canigo 2-6 bajos**  
**Hospitalet de Llobregat**  
**08901 Barcelona**  
**Spain**  
**T +34-932 60 04 00**  
**F +34-932 60 23 14**  
**medi@mediespana.com**  
**www.mediespana.com**

**medi France**  
**Z.I. Charles de Gaulle**  
**25, rue Henri Farman**  
**93297 Tremblay en France Cedex**  
**France**  
**T +33-1 48 61 76 10**  
**F +33-1 49 63 33 05**  
**infos@medi-france.com**  
**www.medi-france.com**

**medi Hungary Kft.**  
**Bokor u. 21.**  
**1037 Budapest**  
**Hungary**  
**T +36 1 371-0090**  
**F +36 1 371-0091**  
**info@medi.hu**  
**www.medi.hu**

**medi Nederland BV**  
**Heusing 5**  
**4817 ZB Breda**  
**The Netherlands**  
**T +31-76 57 22 555**  
**F +31-76 57 22 565**  
**info@medi.nl**  
**www.medi.nl**

**medi Polska Sp. z o.o.**  
**Zygmunta Starego 26**  
**44-100 Gliwice**  
**Poland**  
**T +48-32 230 60 21**  
**F +48-32 202 87 56**  
**info@medi-polska.pl**  
**www.medi-polska.pl**

**medi Bayreuth Unipessoal, Lda**  
**Rua do Centro Cultural, no. 43**  
**1700-106 Lisbon**  
**Portugal**  
**T +351-21 843 71 60**  
**F +351-21 847 08 33**  
**medi.portugal@medibayreuth.pt**  
**www.medi.pt**

**medi RUS LLC**  
**Business Center NEO GEO**  
**Butlerova Street 17**  
**117342 Moscow**  
**Russia**  
**T +7-495 374 04 56**  
**F +7-495 374 04 56**  
**info@medirus.ru**  
**www.medirus.ru**

**medi Trading (Shanghai) Co. Ltd.**  
**Room 105, Building No. 4,**  
**Lane 500 ShengXia Road**  
**Pudong New District**  
**201210 Shanghai**  
**People's Republic of China**  
**T: +86-21 61761988**  
**F: +86-21 61769607**  
**info@medi.cn**

**Medi Turk Ortopedi Medikal Ic ve Dis Tic. AS**  
**Kültür mah. çaldıran sok.14/3**  
**06420 Çankaya/Ankara**  
**Turkey**  
**T +90 312 435 20 26**  
**F +90 312 434 22 82**  
**info@medi-turk.com**  
**www.medi-turk.com**

**medi UK Ltd.**  
**Plough Lane**  
**Hereford HR4 0EL**  
**Great Britain**  
**T +44-1432 37 35 00**  
**F +44-1432 37 35 10**  
**enquiries@mediuk.co.uk**  
**www.mediuk.co.uk**

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#### Important notes

This medical product is made for single patient use only. If the support is used for the treatment of more than one patient or is improperly modified, the product liability of the producer lapses according to the Medical Product Law. If you feel too much pain or an unpleasant feeling when wearing the support, take it off at once and contact your doctor or your specialist provider.

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# medi SAS® 15

## Intended purpose

The medi SAS 15 is a shoulder abduction support. The product is exclusively to be used for the orthotic fitting of the shoulder and only on intact skin or if any wounds have been properly covered.

## Indications

- Conservative/postoperative in acute/chronic bursitis
- Conservative immobilisation of a first-time shoulder dislocation
- After removal of calcium deposits
- Impingement syndrome
- Frozen shoulder following mobilisation under narcosis
- Shoulder prosthesis insertion
- After rotator cuff surgery.

## Contraindications

None known at present.

## Important notes

Only change the angle of abduction on instruction to do so from your doctor or orthotist.

## Fitting instructions

We recommend lying on a table for the application and removal procedure. Pay attention to the following advice regarding the application: Make sure the side release buckles on the front of the cushion as well as the straps are open. Put the shoulder and hip belt on the table.

1. Place the affected arm in the open sling allowing your hand to rest on the ball (image 1).
2. Then, fasten all straps (image 2).
3. Pull the hip belt from behind your back to the front and fasten it (image 3).

4. Pull the shoulder belt over the uninjured shoulder to the front and place your arm in the opening between the shoulder and armpit belt. Now, close the side release buckles.

This product should only be used under medical supervision. Follow instructions of a trained professional, e.g. an orthopaedics technician, when applying for the first time. Qualified and individual adjustment ensures effectiveness and comfort of application. Consult your physician regarding simultaneous usage with other products.

## Care instructions

The product should not be worn in water. Hook and loop fasteners should be closed for washing. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Pillow case, bag and straps can be washed by hand, preferably using medi clean washing agent.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

**Storage instructions**

Please keep the support in a cool, dry place and do not expose it to direct sunlight.

**Material composition**

PU-foam, Polyamid

**Liability**

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

**Disposal**

The product can be disposed of in the domestic waste.

**Your medi team**

wishes you a quick return to full fitness.

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